



For optimal functionality, use Internet Explorer to access. Other browsers will limit some capabilities.

- Log in to: *PACS.ADVANCEDDIAGNOSTICGROUP.COM*
- Enter username and password
Username: _____
Password: _____
- Click on Download Viewer in the upper right corner. Then click Run.
- This will take a few minutes to download. You should see a small pop-up window that says the viewer was downloaded. You can close this window.
- To access patient images, double click on the patient name.
- To access the report, click on the green R to the left of the patient name. (If the R is grey, the report is not available yet. If the R is orange, the report is available but awaiting final Radiologist approval).
- To access key images that have been identified by the Radiologist, click on the orange K to the left of the patient name.
- If you do not see your patient name, go to the bottom of your screen and try checking the box that says Full Scan, then click refresh.
- To search for a patient by name, go to the top of your screen and type the last name of the patient into box that says Patient Name.

Mac/Safari/iPhone users:

- Log in to: *PACS.ADVANCEDDIAGNOSTICGROUP.COM*
- Enter username and password.
- Type in the patient last name in the Search by Patient Box at the top of your screen.
- To access the report, click on the blue box with an R to the right of the patient name. (If you do not see a blue R, the report is not available yet).

**WE ARE HERE
TO HELP**

For more information,
contact your Account
Executive or call
888.234.5575